



Curatronic Ltd.

Ultra Power 3-D System

Your Partner in Healing



Curatron 2000 3-D


State-of-the-Art Ultra-Power Clinical System



Curatron 2000 3-D

The 3-D mattress generates 50,000 microTesla simultaneously over the complete mattress. This guarantees the highest possible penetration through all tissue, cells, and bones deep inside the body. The pulsed electromagnetic field completely envelopes and penetrates the whole body in all 3 axes (length, width and height). This system has been designed for clinical applications where fast results are crucial to both therapist and patient.

How does the 3-D electromagnetic field work?

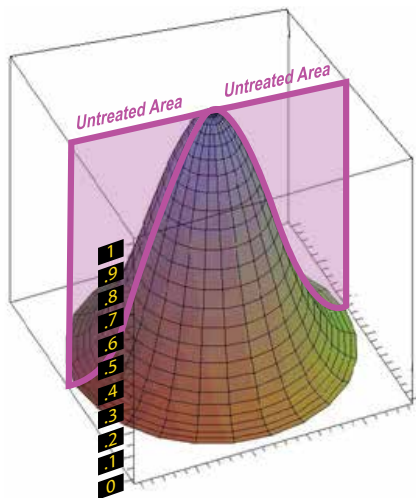

The picture above shows the typical electromagnetic field for one of the flat coils placed inside a therapy mattress. As you move away from the coil the magnetic field diminishes. This means that although the pulsing field will completely penetrate the body, some “dead” areas will occur (as shown in the purple areas of the picture).

The standard Curatron therapy mattress contains 8 large coils inside, which are all activated at the same time. The Curatron mattress has large coils in order to reduce the “dead”


zone effect as much as possible, and to obtain as even a spread of PEMFs as technically possible.

In practice, the “dead” zone effect has no influence on the efficacy of the treatment as people usually do move around somewhat during treatment. Repeated treatments will eliminate this effect almost completely, as nobody lies on the mattress in exactly the same way as in previous treatments.

3-D PEMF Bath

The person is treated with very powerful PEMF's, which can easily be felt with the test magnet, far above the mattress. The whole body is literally “bathed” inside and out by these very strong pulsed electromagnetic fields. This strong PEMF field is still very powerful 20 inches (50 cm) above the complete mattress area.

The body is bathed in a 3-D pulsed electromagnetic field

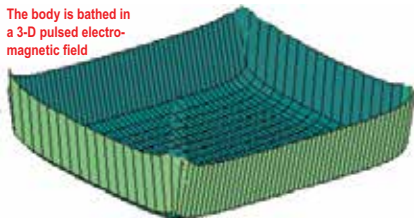

The 3-D ultra-power Curatron system is superior for virtually instantaneous pain relief and treatment of arthritis, osteoporosis, fibromyalgia and other disorders.


A very effective 3-D whole body treatment is assured and patients have no problem relaxing completely on top of the 3-D mattress.




Butterfly Coil for a maximum focused energy field of 160,000 microTesla

Easy to Use Software for PC or MAC platform

Record Patient Data and treatments. ▼

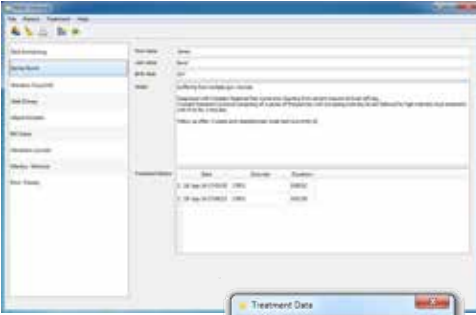

Choose a disorder from the list of pre-programmed settings. ►

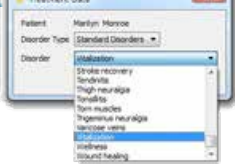

Create a custom disorder with the software, and save your settings for future use. ►



Monitor Patient Progress ▼



Produce Patient Reports ▼

PEMF Software Patient Report

Patient Details

Name: James Bond

Birth date: 007

Notes:

Suffering from multiple gun wounds.

Diagnosed with Complex Regional Pain Syndrome resulting from severe trauma at lower left leg.

Created treatment protocol consisting of a series of frequencies with increasing intensity levels followed by high intensity local treatment with 8 Hz for 3 minutes.

Follow up after 3 weeks and repeated pain scale test (currently 8).

Date	Disorder	Duration (HH:MM:SS)
18-Sep-14 17:03:30	CRPS	0:00:52
18-Sep-14 17:00:15	CRPS	0:02:39

Total treatment time: 0:03:31

Software package included with Curatron 3D system

- All therapy parameters are under direct command and control of the PC program running under Windows or Mac.
- User-friendly database for demographic patient data, including treatment follow up.
- Therapy settings can be selected from a database containing an extensive list of almost one hundred “ready for use therapy protocols.”
- In addition, therapy parameters can be set manually and saved in the Custom Disorders database and then run automatically.
- Free notes editing.
- Full PEMF report printing, including treatment history data, treatment time and accumulated treatment time.
- Automatic USB communication port detection for very easy installation without basic computer knowledge.



2000 3-D Specifications

2000 3-D System	Optional Applicators	Maximum Intensity
3-D Mattress (included)	26"x71" (65cm x 180cm) 3.2" (8cm) thickness	50,000 microTesla
Energy Pad	20"x28" (50cm x 70cm)	40,000 microTesla
High Energy Coil (A)	8"x8" (20cm x 20cm)	70,000 microTesla
Very High Energy Coil (B)	8"x8" (20cm x 20cm)	100,000 microTesla
Butterfly Coil	8"x8" (20cm x 20cm)	160,000 microTesla
Probe	6" x 1" (15cm x 2.5cm)	10,000 microTesla

Two systems are available: System A runs the 3-D mattress, or System B which allows two high power mattresses (25,000 microTesla) to be run simultaneously. System B will have a longer ordering time.

Each organ in the body has cells which operate at different frequencies. Curatronic® conducts research into specific cell frequency and has preprogrammed its devices to stimulate cells at their optimum frequency related to specific disorders. The following is a list disorders and syndromes which the Curatronic® 3D has preprogrammed settings for PEMF treatment on its devices:

Programmed Disorders:

- Acne
- Alzheimer's
- Ankylosing Spondylitis
- Anxiety
- Arthritis
- Atherosclerosis
- Asthma
- Athletic fatigue
- Backache
- Brachial neuralgia
- Bronchitis
- Burns
- Bursitis
- Carpal tunnel syndrome
- Cervical arthritis
- Cervical myalgic headaches
- Cervico brachialgia
- Coxarthrosis
- Coxitis
- Complex regional pain syndrome
- Contusions

- Decubitus
- Delayed fracture healing
- Dental treatment
- Dermatitis
- Dislocations
- Epicondylitis
- Facial neuralgia
- Fibromyalgia
- Fracture healing
- Gonarthrosis
- Herpes
- Hallux valgus bursitis
- Heart
- Intercostal neuralgia
- Intramuscular fibrosis
- Internal injuries
- Irritable bowel Syndrome
- Kidney resonance
- Liver resonance
- Low back pain
- Metabolic deficiency
- Migraine
- Migrainous neuralgia
- Morbus bechterew
- Morbus Sudeck
- Neuritis
- Neurosclerosis
- Neurosis
- Neurovegetative somatic symptoms
- Oedema
- Osteoarthritis
- Osteoporosis
- Osteomyelitis
- Pain therapy
- Pancreatitis
- Periarthritis
- Phlebitis
- Phobia
- Post radiotherapy treatment
- Prostatitis
- Prostate hypertrophy
- Psuedo arthrosis
- Psoriasis
- Relaxation
- Relaxation EEG waves
- Retinitis
- Rheumatoid arthritis
- Sacral arthritis

- Scars
- Sedation
- Sinusitis
- Sleep disturbances
- Spastic colon
- Stomach relaxation
- Stress
- Stroke recovery
- Tendinitis
- Thigh neuralgia
- Tonsillitis
- Torn muscles
- Trigeminus neuralgia
- Varicose veins
- Vitalization
- Wellness
- Wound healing

Brain Disorders:

- Alzheimer's
- Depression
- Neuropathic pain
- Parkinson
- Post traumatic stress
- Smoking cessation

Curatronic is the leading manufacturer of PEMF treatment systems which has proven itself as the standard to meet in both performance and long lasting effects.

www.electro-magnetic-therapy.com
info@curatronic.com



Curatronic Ltd.

www.curatronic.com www.curatronic.blogspot.com
www.pemft.com www.facebook.com/PEMFT
www.curavet.com www.youtube.com/user/Curatron/videos



Disclaimer: Pulsed electromagnetic field therapy (PEMF) has been approved by health authorities as medical therapy for human applications. In the European Community the Curatron devices are certified as medical devices according to the Medical Device Directive 93/42/EEC. The devices are manufactured according to ISO 13485 for Good Manufacturing Practice. In the United States of America the Curatron devices have not been cleared by the FDA. The information provided on this website is not medical advice. Health Canada has not sanctioned PEMF devices, however, PEMF technology is used by hospitals throughout Canada for certain applications. The products marketed on the website and in the brochures are not intended to diagnose, treat, cure or prevent any disease. Readers are encouraged to obtain the help, services and recommendations of doctors and other licensed medical practitioners.